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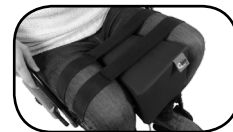
### MUÑEQUERAS

### SUJECIONES

**ATTENTION: QUALIFIED STAFF MUST ALWAYS SUPERVISE THE INSTALLATION.**  
Incorrect use of any product may cause severe lesions or death.

**PRODUCT SELECTION:**

Take a look at our Ubiotex catalog to help you choose the correct product that fit the individual needs of patients.

**PREVIOUS RECOMMENDATIONS FOR USE:**

- A complete clinical examination of the patient (symptoms and needs) must be carried out.
- The consent of the patient or legal guardian is required before using the product. The use of sling products must be explained to the patient or guardian in question to ensure their cooperation.
- Sling products must be used according to the Individualized Care Plan (ICP) of the patient in question. ICP constitutes an evaluation carried out by an interdisciplinary team which may include (without any restriction): Occupational or physical therapy OT, Medicine and Social Services. ICP Plan should include recommendations about restorative nursing, patient's release and pain's prevention.

**NOTE:** Considering that patient's behavior is not 100% predictable, no product is 100% infallible. Patient's safety requires a regular re-evaluation and supervision. A product that has worked in the past may be inappropriate if mental and physic conditions of the patient have been modified. NEVER use an unsafe product. Please contact the responsible medical authority if you have questions about patient's safety.

**ADDITIONAL WARNINGS:**

1. A regular supervision of immobilization process of the patient must **ALWAYS** be carried out by qualified staff. A constant supervision should be required to:
  - Patients with an aggressive and agitated behaviour.
  - Patients who are in danger of swallowing its own vomit. This fact includes patients in a supine position or being unable to seat by themselves.
  - Be ready to react when a danger sign occurs. These patients require both a regular examination and an evaluation of their physical and psychological status.
2. This product must **NEVER** be modified nor repaired. The condition of the product must **ALWAYS** be verified before using it (emergence of flaws or broken pieces, orifices, cuts, deteriorations, dysfunctions with the magnet or Iron Clip closures, etc.). Do **NOT** use dirty or damaged products. Therefore, this fact may cause severe lesions or even death may occur. Damaged products must be disposed of.
3. Straps must **ALWAYS** be fixed to an immobile part of the bed or the chair's edge out of reach of patients (by making quick loops or using buckles). These kinds of slings provide better release process in case of accident or fire risk. Straps mustn't press or slip. If this occurs, the patient can slip from the chair or bed (emergence of severe lesions or suffocation). Removing slings from patients constitute an important activity in case of fire risk or catastrophes in facilities. Straps can be cut with scissors in case of emergency.
4. Ubiotex products must **NEVER** be used on toilets, chairs or furniture not allowing its correct use as indicated in the Directions for use.
5. Ubiotex products must **NEVER** be exposed to flames, fire, smoking materials or high heat sources. Some products can melt or get burned. The smoking/ no smoking policy must be strictly implemented.
6. **NONE** Ubiotex product can be used as a security belt in a vehicle in motion. The design of Ubiotex products doesn't support collisions or sudden stops.

⚠️**WARNING!!** Try zippers or closures (buckles, magnet or iron clip) before each use. Dispose of the product if it doesn't work correctly.

**STORAGE AND MANIPULATION:**

- This product has been designed to be used in regular interior areas, hospitals, etc.
- This device can be stored at ambient room temperatures with normal humidity levels.
- Avoid humidity excess or humidity conditions that may damage the product's materials.

## WHEELCHAIR RESTRAINTS

EN

### DESCRIPTION

The number of people using this kind of belts continues to grow (aged people, patients with spinal injuries related to different accidents, disabled persons...). In this way, the use of wheelchairs is considerably growing to provide these patients a current life that fits the actual lifestyle. Wheelchairs provide a greater mobility and freedom to those people in question. These patients can't maintain an orthostatic position in the chair because they can't control the low part of their bodies due to involuntary movements. In other cases, the patient refuses to keep seated in the wheelchair the time required.

Our company UBIOTEX has developed a huge range of products indicated to patients who must remain in a correct seated position (see image) and also to prevent the emergence of bedsores or current falls. In this way, the daily activity of health professionals is notably improved. In the same way, there are other kinds of pathologies which entail the need of using mechanical restraints to immobilize all the body parts of the patient in general. This product is usually used in extremes situations with the purpose of controlling problematic behaviours which involve a high risk to patients, to other patients and also to health professionals.

Therefore, our company UBIOTEX has developed a huge range of products indicated to immobilize the patient if necessary.

### INDICATIONS

As previously mentioned, the use of this kind of product is connected both to the placement and to the correct immobilization process of the patient if necessary. In other words, the product is recommended to geriatric patients with motor disabilities, senile dementia, Alzheimer, cerebral paralysis, spinal injuries and to any other kind of patients who must remain in a correct seated position. Furthermore, the product prevents the emergence of bedsores or current falls and also improves the daily activity of health professionals in question.

### CLASSIFICATION

UBIOTEX BED RESTRAINTS are considered a type I medical device (rule 1). The conformity evaluation was carried out by following the indications in Annex VII (Declaration CE of Conformity) of the Directive 93/42/CEE and the Directive 2007/47.

In accordance with Annex IX of the Directive 93/42/CEE and its subsequent modifications, the health product is defined as:

- Non-Invasive Health Product

**Rule 1.** – All non-invasive products are included in the Type I category unless one of the following rules is applicable.

### MODELS

#### 1.-WHEELCHAIR / ARMCHAIR ABDOMINAL RESTRAINT BELT

The main function of this restraint belt is to prevent the patient from leaning, sliding or even suffering lesions due to bad postures. Therefore, the product helps the patient to keep a correct seated position. This product is available with padded or breathable material.

#### **A.- ABDOMINAL**

#### **B.- ABDOMINAL RESTRAINT WITH STRAPS:**

In contrast with the abdominal restraint belt, this product provides a greater fastening because the user remains upright (without forward body inclinations).

## **WHEELCHAIR RESTRAINTS**

**EN**

### **2. - WHEELCHAIR / ARMCHAIR PERINEAL RESTRAINT BELT**

The main function of this restraint belt is to prevent the patient from leaning, sliding or even suffering lesions due to bad postures.

Therefore, the product helps the patient to keep a correct seated position by providing an extra support point (placement of a strap in the perineal zone until the back of the patient) to prevent this one from sliding.

This product is available with padded or breathable material.

#### **A. - PERINEAL**

#### **B. - PERINEAL WITH STRAPS:**

In contrast with the perineal restraint belt, this product provides a greater fastening because the user remains upright (without forward body inclinations).

### **3. - WHEELCHAIR / ARMCHAIR VEST**

The main function of this restraint belt is to prevent the patient from leaning, sliding or even suffering lesions due to bad postures. Therefore, the product helps the patient to keep a correct seated position. This product is recommended to those patients who need a large support of the rib cage due to upper limb injuries or to excessive mobility.

This product is available with padded or breathable material.

#### **A. - ABDOMINAL VEST**

#### **B. - PERINEAL VEST:**

In contrast with the abdominal vest, this product provides a greater fastening because the user remains upright (without forward body inclinations from the wheelchair/armchair).

### **4. - WHEELCHAIR / ARMCHAIR DOUBLE PROTECTION VEST**

In contrast with the normal vest, this product disposes of a frontal opening which provides a greater freedom feeling and also fits the patient's anatomy.

This product is available in polyurethane material (light blue or black).

#### **A. - ABDOMINAL DOUBLE PROTECTION VEST**

#### **B. - DOUBLE PROTECTION PERINEAL VEST:**

In contrast with the abdominal vest, this product provides a greater fastening because the user remains upright (without forward body inclinations from the wheelchair/armchair).

### **5. - PADDED ABDUCTOR CHAIR RESTRAINT BELT**

This product has been specially designed to prevent the patient from sliding of the wheelchair or armchair thanks to its fastening system in the groin zone.

This product offers multiple solutions for instability problems due to functional limitations (physical or psychic causes).

In this way, the product disposes of an opening when the lower limbs are in abduction and therefore the product prevents the emergence of frictions and infections in the zone.

The use of this product is recommended when the hip joint article must remain in a permanent abduction position.

### **6. - ABDUCTOR CHAIR CUSHION**

The posture is notably improved. The product helps the patient to keep the legs and knees apart.

Our company recommends this product to patients who must keep both legs in an abduction position.

(MEASURES: 20 long base; 10 short base; 24 length y 15 thickness)

## WHEELCHAIR RESTRAINTS

EN

### 7.- UNIVERSAL BATHING CHAIR RESTRAINT BELT

The product prevents falls while the patient carries out the daily hygiene processes and also ensures an effective fasten to this one.

This product is compatible with all chair's models.

### 8.- HEEL RESTRAINT STRAPS

Our company recommends this product just in cases where heels must be attached to the chair. The mobility of the heels is reduced because of the great attachment of the product (relaxed or tight way). Additionally, the use of this product is recommended in cases where heels must be placed at waist level (due to bad postures, etc.)

### 9.- PADDED CHAIR LEG REST STRAP

This product is a comfortable accessory that improves the wheelchair's quality of use. In this way, the user will find a more comfortable feet position thanks to the footrest.

### 10.- WRISTBANDS

A.- SUAPEL WRISTBAND - Strap closure.

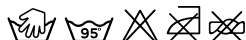
B.- WRISTBAND (PADDED, POLYPROPYLENE AND CANVAS) - Magnet or IronClip closure.

C.- SUJETAX WRISTBAND - Buckle closure.

### CLEANING/DISINFECTION

PADDED AND BREATHABLE MATERIAL: Hand and machine washable at 40°C. Do not use abrasive products. Dry in a horizontal position. Do not expose to direct heat sources as heaters.

CANVAS MATERIAL: Hand and machine washable at 95°C. Do not use abrasive products. Do not iron.

POLYPROPYLENE MATERIAL: Hand and machine washable at 95°C. Do not use abrasive products. Do not iron.

POLYURETHANE MATERIAL: Hand and machine washable at 110°C. Do not use abrasive products. Dry in a horizontal position. Do not exposure to direct heat sources as heaters.

SANITIZED MATERIAL: Hand washable with cold water. Do not use abrasive products. Use soft programs in washing machine. Dry in a horizontal position. Do not iron. Do not use bleach. Do not dry clean.

## WHEELCHAIR RESTRAINTS

EN

### SECURITY

The product must fit the necessities of the user and also the dimensions of the chair/armchair where is going to be used.

**ATTENTION:** The professional supervision from an expert is recommended.

### PRECAUTIONS

Do not use caustic, abrasive, solvent or sharp objects in direct contact with the product.

Test regularly the product's deterioration (orifices, cracks, surface's modification, etc.) to verify the correct function of its technical properties.

Avoid prolonged exposure to intense heat sources or flames.

### MATERIAL'S PRECAUTIONS

Avoid prolonged exposure to liquids and humidity conditions with the product.

### MAINTENANCE

Store the products preferably in a flat position and protected against sunlight and humidity.

### RECYCLING

Do not burn nor throw away in the nature, outside the correct places for this action.

### GUARANTEE

The guarantee period of the products is two years from the date of the original purchase.

This guarantee does not cover the regular deterioration of the product and does not substitute legal guarantees.

Please contact the Quality Department of the company to any incident that may occur to correctly resolve it.

### LIFETIME OF THE PRODUCT

The regular lifetime of the products is about a year.

### LEGEND AND LABEL SYMBOLS

	Maker.
	Serial number.
	Catalogue number.
	Read the instructions for use first.
	Certificate of conformity with Directive 93/42 / EEC
	Caution. Read the instructions for use first.

		MODEL	TISSUE	CLOSURE	REF.	WIDE	INSTRUCTIONS	
BELTS	ABDOMINAL	CHAIR	Padded	Buckles	305502	15 cm	<p><b>(A)</b></p> <p>1. Place the abdominal strap on the abdominal area by placing the product behind the chair/armchair. Attach the straps to the closure system.</p> <p>2. If the belt disposes of a buckle closure system, the straps must be attached through the closure buckle. This strap must be adjusted to the waist of the patient without putting pressure on the zone.</p> <p>If the belt disposes of a magnet or iron clip closure system, the straps must be attached through the hooks of each strap (depending on the bottom colour). Close the bottom once it has been placed.</p>	
				Magnet	304300			
				Ironclip	304310			
		Breathable	Buckles	305550	12 cm			
				305551	15 cm			
		ARMCHAIR	Padded	Buckles	305600	12 cm		
					305601			
	Magnet				304350			
	Ironclip		304360					
	Breathable	Buckles	305651					
	PERINEAL	CHAIR	Padded	Buckles	305501	15 cm		<p><b>(B)</b></p> <p>1. Follow the previous steps mentioned at A.</p> <p>2. Place the perineal strap between the legs. In this way, the perineal strap has to be placed behind the chair or armchair under the user. Attach the perineal strap to the hook of the abdominal strap.</p>
				Magnet	304302			
				Ironclip	304312			
		Breathable	Buckles	305553				
ARMCHAIR		Padded	Buckles	305603				
				Magnet	304352			
				Ironclip	304362			
Breathable	Buckles	305653						
Children	Buckles	305520						

		MODEL	TISSUE	CLOSURE	REFERENCE	INSTRUCTIONS
BELTS WITH STRAPS	ABDOMINAL	CHAIR	Padded	Buckles	305020	1. Follow the previous steps mentioned at: A. 2. Place the straps on the shoulders of the user by placing the product behind the chair/armchair. 3. Attach the straps to the hook of the abdominal strap in buckle closure systems. Attach the straps to the abdominal strap by using the hooks and fasten the product with the closure system, employed: magnet or iron clip.
				Magnet	304301	
				Ironclip	304311	
		Breathable	Buckles	305552		
		ARMCHAIR	Padded	Buckles	305602	
				Magnet	304351	
	Ironclip			304361		
	Breathable	Buckles	305652			
	PERINEAL	CHAIR	Padded	Buckles	305010	1. Follow the previous steps mentioned at B. 2. Place the straps on the shoulders of the user by placing the product behind the chair/armchair. 3. Attach the straps to the hook of the abdominal strap in buckle closure systems. Attach the straps to the abdominal strap by using the hooks and fasten the product with the closure system, employed: magnet or iron clip.
				Magnet	304303	
				Ironclip	304313	
			Breathable	Buckles	305554	
ARMCHAIR		Padded	Buckles	305604		
			Magnet	304353		
			Ironclip	304355		
		Breathable	Buckles	305654		

		MODEL	TISSUE	CLOSURE	SIZE	REF.	INSTRUCTIONS	
ABDOMINAL	CHAIR	Padded	Buckles	S		305021	<p>1. Place the vest on the chest of the patient by placing the abdominal and the shoulder straps behind the chair or the armchair. Attach the straps (depending on the closure system in question). The buckle system is the most common closure system.</p> <p>2. If the restraint belt disposes of a buckle system the straps must be attached to the securing buckle. Adjust the product to the waist of the patient without excessive pressure. If the restraint belt disposes of a magnet or iron clip closure system, the straps must be attached to the hooks of each strap (depending on the colour of the bottom). Close the bottom.</p> <p>ATTENTION: Our Company disposes of an abdominal vest model with zipper. In this case, the use of the product is easier (manipulation of the product in the frontal zone thanks to the zipper).</p>	
				M		305022		
				L		305023		
		Magnet	S		305050			
			M		305052			
			L		305054			
	Breathable	Buckles	S		305560			
			M		305561			
			L		305562			
		Buckles	S		305612			
			M		305611			
			L		305610			
ARMCHAIR	Padded	Buckles	S		305662			
			M		305661			
			L		305660			
	Breathable		S		305662			
			M		305661			
			L		305660			
VESTS	PERINEAL	CHAIR	Padded	Buckles	S		305011	<p>1. Follow the previous steps mentioned at E.</p> <p>2. Place the perineal strap between the legs. In this way, the perineal strap has to be placed behind the chair or armchair under the user. Attach the perineal strap to the hook of the abdominal strap.</p>
					M		305013	
					L		305015	
			IronClip	S		305017		
				M		305018		
				L		305019		
		Breathable	Buckles	S		305565		
				M		305564		
				L		305563		
			Magnet	S		305640		
				M		305641		
				L		305642		
	ARMCHAIR	Padded	Buckles	S		305615		
				M		305614		
				L		305613		
		Breathable		S		305672		
				M		305671		
				L		305670		
Magnet	Buckles	S		305644				
		M		305645				
		L		305646				
	Magnet	S		305644				
		M		305645				
		L		305646				

MODEL		TISSUE	CLOSURE	SIZE	REF.	INSTRUCTIONS
VESTS WITH ZIPPER	CHAIR	Padded	Buckles	S	305702	1. Place the part of the vest on the torso of the user by closing the zipper and passing behind the chair or armchair the tapes of the abdominal area and the tapes of the shoulder area. Join the tapes depending on the closing system in question. The most common is buckles. 2. If the belt is fitted with the buckle system, attach the straps by attaching the locking buckle and adjust to the waist of the patient without tightening or strangling. If the belt carries the magnet system or Ironclip join the tapes by attaching the flaps of each of the ribbons to the button colour. Once the button is pressed, close the button.
				M	305701	
				L	305700	
	S			305712		
	M			305711		
	L			305710		
ARMCHAIR						

MODEL		TISSUE	CLOSURE	SIZE	REF.	INSTRUCTIONS
DOUBLE PROTECTION VEST	ABDOMINAL	Saniluxe	Buckles	S	305802	1. Place the product on the head and shoulder's zone. 2. Attach the abdominal strap (the shortest one) to the back strap to fasten the user. 3. Attach the frontal abdominal straps (the longest ones) to the back of the chair to correctly fasten the patient in the wheelchair/armchair.
				M	305801	
				L	305800	
	S			305902	1. Follow the previous steps mentioned at H. 2. Place the perineal strap between the legs. In this way, the perineal strap has to be placed behind the chair or armchair under the user. Attach the perineal strap to the hook of the abdominal strap.	
	M			305901		
	L			305900		
PERINEAL						

MODEL		CLOSURE	COLOUR	REF.	INSTRUCTIONS
WRISTBANDS	Suapel	Straps	White	111100	1. Place the suapel strap around the wrist and adjust it with the Velcro. 2. Take the longest strap, place it around the wrist and put it inside the cotter pin to fasten the product. 3. Attach the extra strap to the wheelchair.
			Navy	111200	
			Grey	111202	
	Polypropylene	Magnet IronClip	Black	303100	1. Place the strap of the product around the wrist and adjust it matching the hooks to close it with the magnet or iron clip closure system.
			303104		
	Padded	Magnet IronClip	Blue	303114	
			303305		
	Canvas	Magnet IronClip	Beige	303110	
			303108		
	Sujetax	Buckles	Blue	111210	1. Place the strap around the wrist by putting the towelling strap into the metallic hook. Adjust and attach the towelling strap to the hook to correctly fasten the wristband. 2. Take the polypropylene strap, attach it and adjust it at desired height.

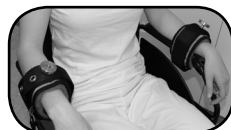
MODEL		TISSUE	CLOSURE	REF.	INSTRUCTIONS
SUPPORTS	ABDUCTOR BELT	Padded	Buckles	307100	<ol style="list-style-type: none"> <li>1. Place the patient comfortably in the chair</li> <li>2. Pass the belt through the abductor area by tightening the straps to fit the contour of the patient's waist, making sure there is no slack and a firm fit.</li> </ol>
	ABDUCTOR CHAIR CUSHION	Saniluxe		705155	<ol style="list-style-type: none"> <li>1. Place the abductor cushion by placing the smallest part on the groin zone. The biggest part has to be placed on the knees. Adjust the product with the Velcro closure system.</li> </ol>
	UNIVERSAL BATHING CHAIR RESTRAINT BELT	Polypropylene		308800	<ol style="list-style-type: none"> <li>1. Place the abdominal strap on the user's abdomen by placing these straps behind the chair or armchair.</li> <li>2. The product disposes of two buckles (one placed in the frontal zone and another in the back) to improve the belt's manipulation.</li> </ol>
	HEEL RESTRAINT STRAPS	Padded		307200	<ol style="list-style-type: none"> <li>1. To place the item correctly, the patient should be placed comfortably in the chair / armchair</li> <li>2. The restraint should pass behind the cuff and tightening the retaining straps to fit the contour of the patient's ankles, making sure there is no slack and a firm grip.</li> </ol>
	PADDED CHAIR LEG REST STRAP			307201	<ol style="list-style-type: none"> <li>1. Place the strap in the upper zone of the footrest of the chair at calf muscle's level.</li> <li>2. Attach the straps behind to adjust and fasten the strap in a correct way.</li> </ol>

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