# Aband

Dear customer, we thank you for your trust in the products of EMO. By choosing an orthotic of the Aband line you have acquired a qualitative medical product. Read these instructions carefully. If you have any questions, please contact your doctor or your specialist.

## NORMATIVE

The medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. Class I medical device, non-sterile, without measurement function.

## DESCRIPTION AND CHARACTERISTICS

PL100 Espaldillera: Composed of two elastic bands that form a ring at each end. Gradual traction: An elastic orthotic in the form of an "eight" that is adjustable with Velcro. PL210 Espaldillera transpirable: composed with two breathable elastic bands in the form of an "eight". Gradual traction: adjustable by Velcro. FJ341 Faja Costal and FJ342 Faja Costal Señora: elastic belt with soft cotton feel. Adaptable to the chest and stabilizes and relieves discomfort. Latex free: hypoallergenic. Due to the fabrication without sewing, overpressure and scratches to the body will be avoided. Weight: minimum thickness. FJ342 Ortec Costal señora: applicable for the lower body to make adjustments to the female anatomy. FJ207 Breathable abdominal band and FJ211 Reinforced breathable abdominal band: made with breathable fabric. Latex free: hypoallergenic. Minimum thickness and fits the waist and the abdominal contour. Wide design that is adjustable with Velcro. FJ211: incorporated rear flexible rods. FJ183 Multi-abdominal band and FJ184 Multi-abdominal band: Made with soft elastic fabric with a cotton feel. Due to its linked but independent bands, adaptation to the morphology of the patient can be facilitated. Wide design that is adjustable with Velcro. FJ184: its height allows treatments on the upper and abdominal thoracic area. FJ160 Abdominal cotton band: made with a soft elastic cotton fabric. The high elasticity of the material makes it possible to adapt the product to the waist and hip. Wide design that is adjustable with Velcro. FJ184. FJ161 Ortec Tubular cotton waistband: Made with a soft cotton and stretchable fabric. Tubular design: adaptable to the patient's anatomy. The high elasticity of the material makes it possible to adapt the product to the waist and hip. PAD80 and PAD115 Hernia pad: Made out of viso-elastic material with a medium density. It could be strapped with Velcro on the inside. SOT10: made with a soft elastic fabric. It is formed by a thoracic band that could be easily attached on the thorax of the patient with the help of Velcro. Due to its anatomical shape,

excellent adaption to the morphology is guaranteed. On each end of the belt there is an incorporated handle to adjust the height on the body.

## EFFECTS

PL100 and PL210: Postural correction of the back (curved back or crooked back). It gives the patient a reminder of a correct posture. It retracts the dorsal column with gradual traction. FJ341 and FJ342: Thoracic fixation. Controls thoracic movements with the help of compression. FJ207 and FJ211: Abdominal restraint. Light lumbar support: FJ183 and FJ184: Thoracic containment. Abdominal restraint. Controls mobility of thoracic. FJ160 and FJ161: Abdominal restraint with therapeutic heat. Accessories PAD80 and PAD115: SOT10: protects and provides safety after thoracic or cardiac surgery during specific situations such as coughing, sneezing or movements. It controls the pain produced during daily movements. The patient should use is the strength of his hands to operate the product.

## INDICATIONS

PL100 Espaldillera and PL210 Espaldillera transpirable: Mild back pain due to an incorrect posture and curved back. FJ341 Faja Costal and FJ342 Faja Costal señora: costal syndrome. Crack in back. Fracture of intercostal muscles. Used after surgery. FJ207 Breathable abdominal band and FJ211 Reinforced breathable abdominal band: for small and medium events. Muscle hypotonia. Post-surgical and postural rehabilitation. FJ183 Multi-abdominal band and FJ184: Post-surgical, event rations, thoracic contusion. Injuries to the abdominal and thoracic muscles. FJ160 and FJ161 Abdominal cotton band: little events. Post-surgical. Accessories PAD80 and PAD115: Along with an Aband elastic orthosis: umbilical hernia and abdominal hernia. Used after thoracic or cardiac surgery during specific situations of daily movements. SOT10: Used after thoracic or cardiac surgery during specific situations of daily movements.

## WASHING AND MAINTENANCE

Hand wash with warm water (maximum 30 degrees) and neutral soap. The Velcro on the products must be kept closed during the washing. Dry at room temperature. Do not iron the garment. Contact with ointments, creams, alcohols and solvents may deteriorate the garment. Do not use bleach or dry clean. Do not expose the orthotic to a direct heat source by drying. Do not expose to a direct heat source. Dry and store the product at room temperature, preferably in the original package.

## INSTRUCTIONS

PL100 Espaldillera: Put the arms through the rings that are formed at the ends of each piece. Place and cross the bands through the center of the back. Close and adjust tension with the Velcro. PL210 Espaldillera transpirable: put the arms through the rings formed by the bands of the orthotic. Close and adjust the garment at the front with the Velcro closure. FJ341 and FJ342: Hold the strip by its ends with both hands. Place the belt over the costal area at the correct height. Adjust the tension and close the garment with the Velcro closure once obtained the right compression. FJ207, FJ211, FJ160, FJ161, FJ183 and FJ184: hold the strip by its ends with

both hands. Place the belt on the abdominal area at the correct height. Adjust the tension and close the garment with the Velcro closure once obtained the right compression. SOT10: Place the orthotic around the patient's chest by adjusting the height using the straps. Adjust the measurement of the orthotic with the Velcro. The orthotic must be adjusted in a way that the patient can grab both handles with only one hand and with sufficient power. Subsequently, fix the elastics at the two ends of the orthotic to make it possible to grab the two handles of the garment. When wearing the orthotic the user must be able to grab both handles with one hand with sufficient power.

## PRECAUTIONS

Follow the manufacturer's directions for proper orthosis placement. Follow the instructions or recommendations of the prescriber or healthcare personnel. For a personalized adaptation, consult a specialist. If there is ongoing pain, it is advisable to contact a healthcare professional. The used material is hypoallergenic but if there is any allergic reaction or irritation remove the product and consult with the prescribing doctor. In case of discomfort caused by excessive sweating we recommend to wear a cotton t-shirt. The user should position itself in such a way that the compression is not excessive. In case of doubt about the size, choose the bigger size. Only use during active periods. Reduce abdominal pressure during daytime breaks or after meals. Remove during the night. The correct choice of the appropriate size for each patient is essential to obtain the highest degree of therapeutic efficacy. The adjustments of the orthotic should be made individually. The orthotic should be used for the purpose it is prescribed for. Do not expose the orthotic to a flame, heat source or high temperatures.

## WARRANTY

Especialidades Médico Ortopédicas s.l., guarantees that all its products, as long as they have not been manipulated or altered, always are within the framework of the legal terms. Products whose characteristics have been altered due to misuse are not included in this warranty.

## NOTICE

To dispose of the product, use the solid waste container. For the disposal of the container, depending on its material, use the paper / cardboard container or the plastic container. In any case, you must strictly comply with the regulations of the country. Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and / or the patient are established.

## CERTIFICATES

All the used materials have been tested, comply with the applicable European regulations, are of high quality and offer maximum safety

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