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BEFORE USE

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.

INTENDED PURPOSE

Raised toilet seat to assist those who need to sit down and stand up from the toilet easier. Models are available with / without a lid to best meet the user's needs.

ASSEMBLY

Insert the side brackets:

- 1- Turn the toilet lift upside down and place it on a flat surface.
- 2- Insert the brackets into the side guides. Push lightly and turn the screw wheel clockwise to fit the brackets into the toilet riser.

Note: Don't tighten the brackets all the way at this stage, otherwise you won't be able to position the toilet lift on the toilet you want to install it on.

3- Place the toilet lift in the desired toilet. Now tighten the side adjustments you fitted earlier. Use the side adjustment wheel to tighten them until the toilet lift is securely fitted.

Note: Do not overtighten to avoid breaking the parts. The side locking system is designed for easy, tool-free adjustment.

MAINTENANCE AND CLEANING

- Clean regularly by hand with a damp cloth
- Do not use aggressive cleaning agents
- Do not use abrasive cleaning agents
- Do not use cleaning agents with a high chlorine or vinegar content
- Check screws regularly and retighten if necessary.
- The product must not be subjected to temperatures above 60 °C for long periods (e.g. in saunas, etc.).

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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