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Cane

### Main components

The product is composed of hand grips, components and pad of the supporting leg, adjusting and etc

### Intended Use

The Cane is a walking aid for a lower limb disabled patient. Its main function is to help lower limb fracture patient to recover.

## CONTRAINDICATION

None

## WARNINGS

1. For safety, these precautions must be taken when using this product. Before each use make sure.
  - 1.1 Cane is adjusted to correct height.
  - 1.2 Spring buttons are fully engaged, and the locking collar is tight.
2. Tips are in good condition. To preserve the life of the tips, do not drag or rock.
3. The cane when walking, replace tips immediately when worn.
4. Hand grips are attached securely and does not slip.
5. Do not exceeded the weight capacity of the individual cane. Injury to the user or the damage to the cane may occur. All listed canes have a weight capacity of 150Kg.
6. This cane is designed to be a working aid not to support the full body weight of the user. Use only as instructed by your healthcare provider. Always use extreme care when using a cane. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path, Failure to follow the above instructions may result in serious injury.

## USAGE METHOD

1. When using a cane for stability, you may grip it in either hand.
2. Hold the cane in the opposite hand of the affected leg and move the cane unison with the affected leg. Each time you step with the affected leg, also move the cane to give you support as you walk. When you step forward with the unaffected leg, keep the cane in place for support.
3. Your healthcare provider might recommend a specific walking pattern.
4. Push button adjustable.
  - 4.1 Loosen the locking collar.
  - 4.2 With your shoes on, and your arm down at your side, you wrist should be even with the handle of the cane. Engage the spring button into the locking collar.
  - 4.3 Retighten the locking collar.
5. Quad canes
  - 5.1 Adjusting for right- or left-hand use
  - 5.2 Loosen the locking collar
  - 5.3 Depress the spring bottom
  - 5.4 Rotate the handle ½ turn engaging the spring button in a hole that provides correct height.
  - 5.5 Retighten the locking collar.

## CLEANING

Clean method: use wet cloth to wipe

Clean the grips with a mild soap, detergent or household cleaner, do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen. The tips on this product contain dry natural rubber.

## MAINTENANCE

1. Periodically inspect the cane for damage or loose components.
2. Check tips are in good condition. To preserve the life of the tips. Replace tips immediately when worn.

## WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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