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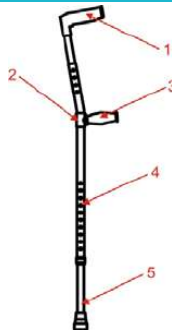
INSTRUCTION FOR USE

Main components

- 1-Arm sleeve
- 2-Support bracket
- 3-Handle
- 4-Height adjustment
- 5-Feet

Intended Use

The Crutches are a mobility aid that transfers weight from the legs to the upper body.

It is used by people who cannot use their legs to support their weight, for reasons ranging from short-term injuries to lifelong disabilities.

INSTALLATION INSTRUCTIONS

Use to keep your weight off your leg or foot.

1. Adjust your crutches before using. Press the spring lock with your finger and adjust to the suitable height. The top of the crutches should reach 2,5 to 5 cm below your armpits while you stand up straight. The handgrips of the crutches should be even with the top of your hip line. Your elbows should bend a little when you use the handgrips. Hold the top of the crutches tightly to your sides and use your hands to absorb the weight. Don't let the tops of the crutches press into your armpits.

2. Walking: Lean forward slightly and put your crutches about one foot ahead of you.

Begin your step as if you were going to use the injured foot or leg but shift your weight to the crutches instead. Your body swings forward between the crutches. Finish the step normally with your non-injured leg. When the non-injured leg is on the ground, move your crutches ahead in preparation for the

next step. **Sitting:** Back up to a sturdy chair. Use your hand to feel for the seat of the chair and slowly lower yourself into it.

3. Make sure the rubber tips on your crutches are tight and stable. If they are worn, change them immediately.

WARNINGS

1. Check whether the grip handle is loose or burr before use to ensure the safety.
2. Check whether the rubber base of the crutch is tight and stable, if it is worn, it should be replaced in time; whether the marble is exposed, and the lock nut is tightened.
3. The walking stick is a walking aid. Please do not use it as a supporting object other than the walking aid.
4. The user's weight cannot exceed 120 KG.
5. Crutches are not suitable for mountain climbing.

CLEANING

Use a damp cloth and neutral soap.

STORAGE AND TRANSPORT

Store in a cool and ventilated place. Transport: avoid strong pressure, direct sunlight, rain or snow.

SHELF LIFE

Under normal conditions of use, the crutches are designed for a service life of at least 10 years.

WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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