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Manual de instrucciones

**Foldable Commode Chair 4-in-1**

Instruction Manual

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**Chaise Percée pliante 4 en 1**

Manuel d'instructions

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## DESCRIPTION

Removable aluminium chair with built-in toilet.

Multifunction chair that offers 4 functions in a single chair, as it can be used as a height-adjustable toilet lift, as a chair with toilet for the bedroom (incorporates a bucket), as a seat for the shower and as an auxiliary support for the toilet.

Designed for people who for some reason have difficulty getting to the toilet. Easy to clean and height adjustable seat.

## INSTALLATION INSTRUCTIONS

1. Adjust the toilet seat to the required height by pressing the marble on the leg tube. Please note that the marbles of all four legs must be positioned at the same height.
2. Lift the seat cover, place the toilet on the seat.
3. When in use, the seat cover can be used as a backrest.
4. After use, close the toilet pan lid and lift it straight up from the toilet pan support.
5. The commode chair can be folded when not in use, so it takes up less space.

6. Remove the backrest and toilet bowl and place them on the toilet to be used as a safety fence.
7. Add an anti-splash bucket, place it on the toilet and use it as a hoist and safety fence.

## MAINTENANCE

1. Routinely clean and disinfect once a day by medical staff or users, and thoroughly disinfect the chair when changing patients.
2. Do not contact with strong acids and alkalis to prevent corrosion.

## WARNINGS

1. Do not use if you detect that any of the parts of the wheelchair are in poor condition.
2. Be sure to adjust all four legs to the same height and check that it is on a flat, stable surface.
3. Do not exceed the maximum weight indicated in the manual and on the product label, which is 130 kg.
4. Make sure that the product is adapted to the characteristics and physical condition of the user.
5. Do not stand on the seat. It is not suitable for use as a mobility aid.

## STORAGE

The product should be stored in normal temperature and humidity and keep dry.

## SHELF LIFE

5 years.

## WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

## WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

## ASSEMBLY

Open the side parts of the structure, lift the central part with the WC support already mounted and fix the two ends to the side parts.

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