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
getting started

Step 1: Determine the height needed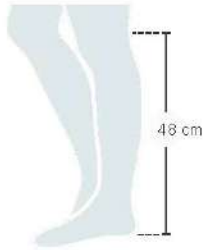

Measure the length of
your lower back leg

Add 5 cm
 $48 \text{ cm} + 5 \text{ cm} = 53 \text{ cm}$

Measure the height of
your toilet bowl

Be sure not to include
the seat and lid

Calculate the difference
 $53 \text{ cm} - 43 \text{ cm} = 10 \text{ cm}$

You need a 10 cm raised
toilet seat!

Step 2: Determine the shape of your toilet



Toilets come in 2 different shapes

- Standard Round
- Elongated

To ensure you are correctly measuring your toilet bowl, measure from the center front to the center of the mounting holes.

parts

- A. Armrest (2)
- B. Armrest holder
- C. Locking mechanism for adjustable holder
- D. Body
- E. Recessed grip (in the lid)
- F. Lid
- G. Fixing screw (2)
- H. Knurled nuts (2)
- I. Rear support
- J. Adjustable holders (2)
- K. Supports (4)
- L. Hygiene openings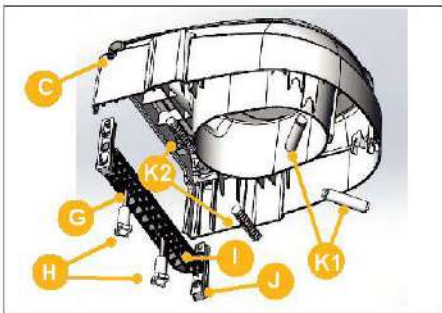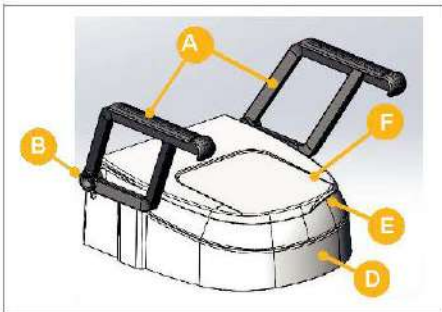


WEIGHT CAPACITY: 350 LB / 158 KG


assembly

1. Remove existing toilet seat and lid assembly
2. Place rear support (I) on toilet bowl and attach at required length based on your toilet shape using fixing screws (G) and knurled nuts (H).


NOTE: There are letter indicators on the rear support (I) marked "E" for Elongated and "S" for Standard to assist with which hole to thread the fixing screws (G) through for your toilet shape.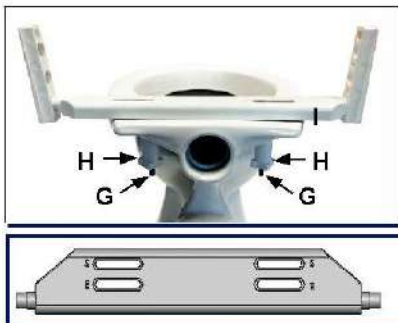


3. Insert adjustable holders (J) onto the rear support (I) at the desired seat height.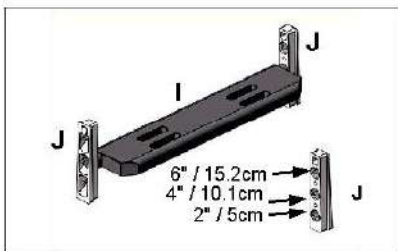

4. Push armrests (A) into the armrest holders (B). If not using armrests, skip this step.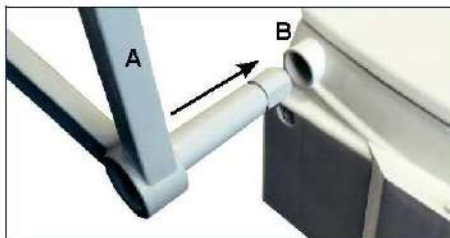

5. Slide body (D) of raised toilet seat onto adjustable holders (J) until you hear a click into place.

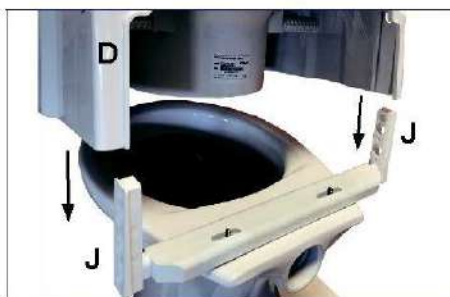height adjustment

The raised toilet seat can be adjusted using the adjustable holders (J) for an elevation of 2" (5cm) 4" (10.1cm) and 6" (15.2cm). If you wish to change the seat height, follow these steps:

1. Push the locking tabs on the right and left under the armrests (A) and pull the body (D) of the raised toilet seat up.
2. Pull out the adjustable holders (J) and set the desired height. To do this insert the rear support (I) into the corresponding hole in the adjustment holder (J). Be sure that the adjustable holders (J) are set at the same height on both the left and right sides.
3. To adjust the height in the front, insert the supports (K) into the corresponding openings until you hear them click into place. Be sure that the supports (K) are set at the same height on both the left and right sides.

NOTE: The supports (K) can be set lower than the rear support (I).

4. Set the body (D) of the raised toilet seat back onto the rear support (I).

lid assembly

The lid is preassembled at the time of delivery but can be replaced if necessary. To do this, turn the body of the raised toilet seat around and press the lid clips at the same time. The lid can then be taken out of the holders.

additional precautions

- Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
- Check for secure fit routinely before using product. User's weight should be centered over toilet seat, not too far forward or too far to one side, as unit may tip.
- Use the arm supports only for assistance. DO NOT attempt to use the arms to support full body weight.
- The lid must not be used as a backrest in the opened position.
- The lid must not bear a load when closed.

maintenance

Recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Products are manufactured with an antimicrobial* agent that helps to minimize the growth of stain and odor-causing bacteria and mold on treated surfaces.

*Antimicrobial properties are built in to protect the products. These products do not protect users or others against bacteria, viruses, germs or other disease organisms.

WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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