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## DESCRIPTION

The product consists mainly of desktop, stand and wheels. The desktop is made of wood or plastic.

## INSTRUCTIONS

1. Take out all parts.
2. Assemble all screw according to the arrow way, and fix them well.
3. Turn on knob to fix or loosen the gear between the small and big table top and adjust the angle if need.
4. Max weight capacity: 20KGS.

5. Add some oil to the movement parts after some time using.

7. Keep in dry place.

## WARNINGS

1. The table should be safeguarded against humidity and should not be exposed to sunlight for long periods.
2. The appliance is not to be used by children or persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction.

## STORAGE

The product should be stored in normal temperature and humidity and keep dry.

## SHELF LIFE

10 years.

## WASTE DISPOSAL

Please dispose of the product after use to comply with local regulations.

## WARRANTY

This product is guaranteed for three years from the date of purchase by the user. The manufacturer will supply replacement parts for any part that has broken due to manufacturing faults. Please contact the distributor from whom you purchased the product in case you need to make a claim. Faults resulting from accidental damage, misuse, alteration or wear and tear are not covered by this guarantee. Please contact the dealer from whom you purchased the product to activate the warranty.

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Foshan S.M.P Co., Ltd
SUNGO Europe B.V.